Instruction for use

EN

The Lola&Lykke Breast Milk Storage Bags are intended to be used for storing breast milk, transporting breast milk and warming up breast milk for feeding the baby.

Important safety information

- The breast milk storage bag is a single-use product. It is not intended to be reused or reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross-contamination.
- Do not thaw frozen breast milk or warm breast milk in a microwave or a pan of boiling water, to avoid loss of vitamins, minerals and other important components and to prevent burns.
- Do not fill bag past the 150ml mark, breast milk expands as it freezes.
- Wash your hands thoroughly with soap and water before touching the storage bag. Dry your hands with a fresh towel or a single-use paper towel.
- Avoid touching the inside of the storage bag.

Storing Breast Milk

- As soon as you are finished pumping, place the breast milk bottle or breast milk storage bag in a cool place like a cool bag if not at home.
- Transfer breast milk bottles or breast milk storage bags to the refrigerator or freezer once you are home.
- When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion. Label bottles/bags with the date of pumping.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37 °C).
- Gently shake the bottle/bag to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

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